## **1. Product Name**

Linelite Shower

## **2. Product Code**

18312

## **3. Colour**

Differing colours.

## **4. Brief Description**

Stand amongst or move through the multi-coloured linelite strands.

## **5. Contents**

1 x Linelite shower, ready assembled.

## **6. Snoezelen® Stimulations**

- Touch
- Sight
- Sound

## **7. Best Use**

Use in a darkened environment with an ultra violet light for best effect.

## **8. Compatible ROMPA® Products**

- 20714 - LED Shadow
- 14531, 14532 - Ultra Violet Light
- 18376, 18377 - Low energy black light bulb

## **9. Starting Up**

- Unpack item
- Dispose of packaging safely and responsibly.
- Install linelite shower. Ensure that the item is located in an area where it is accessible and safe. Consider proximity to doors, walls and hazards.
- Secure safely to the ceiling near a UV light.
- Switch off the main lights and switch on the UV light.
- Enjoy the tactile, visual and audible input with the bells.

## **10. Detailed Description**

These cascading strands, suspended from the ceiling offer multisensory input. They have tactile, visual and audible appeal with bells, and when exposed to UV light, these linelite coloured strands react vividly.

## **11. Safety**

- Always supervise the use of this product.
- For use in dry environments only.
- Do not pull on the strands – they can be safely touched and walked through but not pulled on.
- Do not hang anything heavy from the strands

## **12. Technical Specification**

Weight            1.65kg  
Strands:         140cm long

### **13. Installation**

Suspend the Linelite Shower from your ceiling, considering the following points:

1. The Linelite shower weighs approximately 1.65kg – choose fixings (e.g. a screw-in hook) that are appropriate to the construction of your ceiling and to the weight of the product
2. Length – please see specifications
3. There is a cord on the disc to suspend the Linelite Shower with – this measures 92cm – from the loop at one end to the point where the three cords from the disc meet. The cord can be adjusted – tie a knot to form a loop further down the cord as required.
4. The Linelite Shower looks more effective in a darkened environment with an ultra violet light nearby.
5. The Linelite Shower is not electrical – proximity to a wall socket is not required.

### **14. Care and Maintenance**


- Wipe the strands clean as required using a mild household cleaning product.

### **15. Troubleshooting**

Not applicable

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at [producthelp@rompa.com](mailto:producthelp@rompa.com)

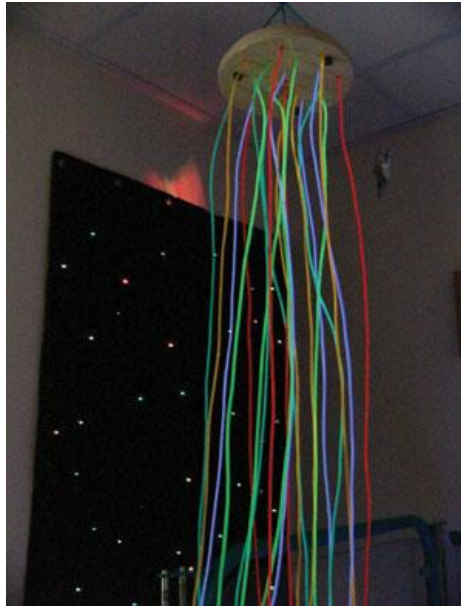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